

Thyroid Stimulating Hormone

Page 1 of 1

Diagnostic Test Information Sheet 03

Test Name: Thyroid Stimulating Hormone

Test code: TSH3

Special Precautions: None

Specimen Handling: 1. Collect specimen in a 5 mL SST Gold top tube.

2. Gently invert tube 5-6 times.

3. Label specimen with:

o full name of patient

o one other unique patient identifier (i.e. DOB and/or

Medical Health Number)

test name

4. Allow to clot upright for a minimum of 30 minutes.

5. Centrifuge for 10 minutes at 1300 RCF.

6. Place specimen in specimen transport bag.

7. Seal bag.

8. Fold requisition and place in sleeve of bag.

Reference Range: 0.32-5.04 mIU/mL

Please note new reference ranges effective Dec 21, 2021

Turnaround Time: Not available on a STAT basis.

Routine: 72 hours

Sample Type: Serum or Plasma

Sample Container: Gold Top Serum Separator Tube or Plasma Separator Tube

Minimum Volume: 1 ml

Collection and Shipping: Refrigerate @ 2-8° C. If sent out to referral lab, send on ice pack.

Method: Beckman Coulter Access 2- Immunoenzymatic Assay

Stability Time: 2 days @ 2-8 C.

Interferences: Hemolysis: avoid hemolyzed samples

Lipemia: **avoid lipemic samples**Bilirubin: no significant interference

Measurement Uncertainty: Available upon request. Please call 604-336-2090 or email

info@novavitalaboratory.com.

Revision Number:	1.5	Effective Date:	Nov 16, 2023